

DataMaster Calibration Logbook Entry Instructions

- All DataMaster Calibrations shall be logged. ^{language}
- A log entry shall be made **each** time the Calibration solution is analyzed. ^{used?}
- A new bottle of Calibration solution should be used for each calibration.
- A previously used Calibration solution may be used if the solution is not depleted. ^{must, shall}
 - If the solution has been open for more than one week, it shall be replaced.
 - If the solution has been analyzed more than twenty (20) times, it shall be replaced.
- The simulator temperature must be confirmed with a NIST traceable thermometer prior to analysis.
- The simulator temperature must read $34.0^{\circ}\text{C} \pm 0.2$